
# MTG Eagle® Board

## Instructions for Use<sup>a</sup>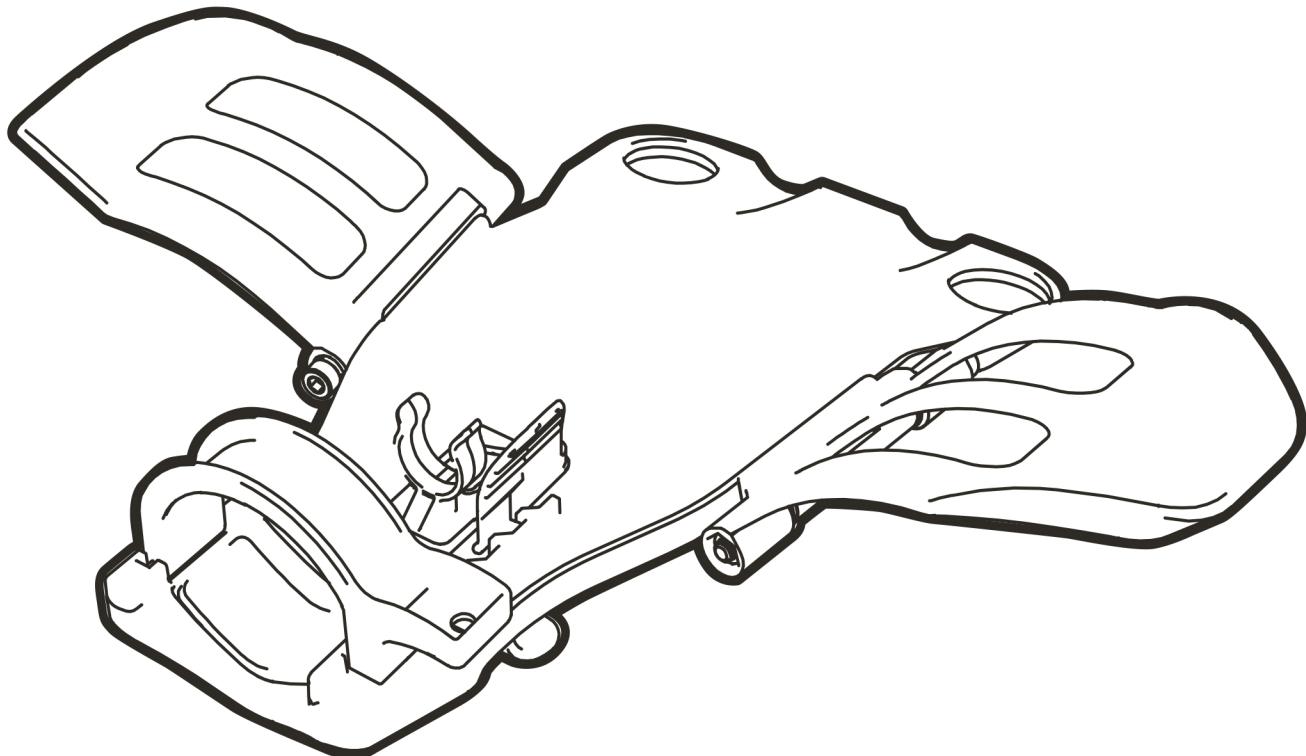

For use with MTG EZ-Gripper® Intermittent Catheters<sup>b</sup>

Medical Technologies of Georgia

15151 Prater Dr., Suite E,

Covington, GA, USA, 30014


01-(770)-788-0763

[www.medtechga.com](http://www.medtechga.com)

PK-1298 REV 03 08/21

# MTG Eagle® Board Kit Contents<sup>a</sup>

#2 Philips head screwdriver not provided.<sup>b</sup>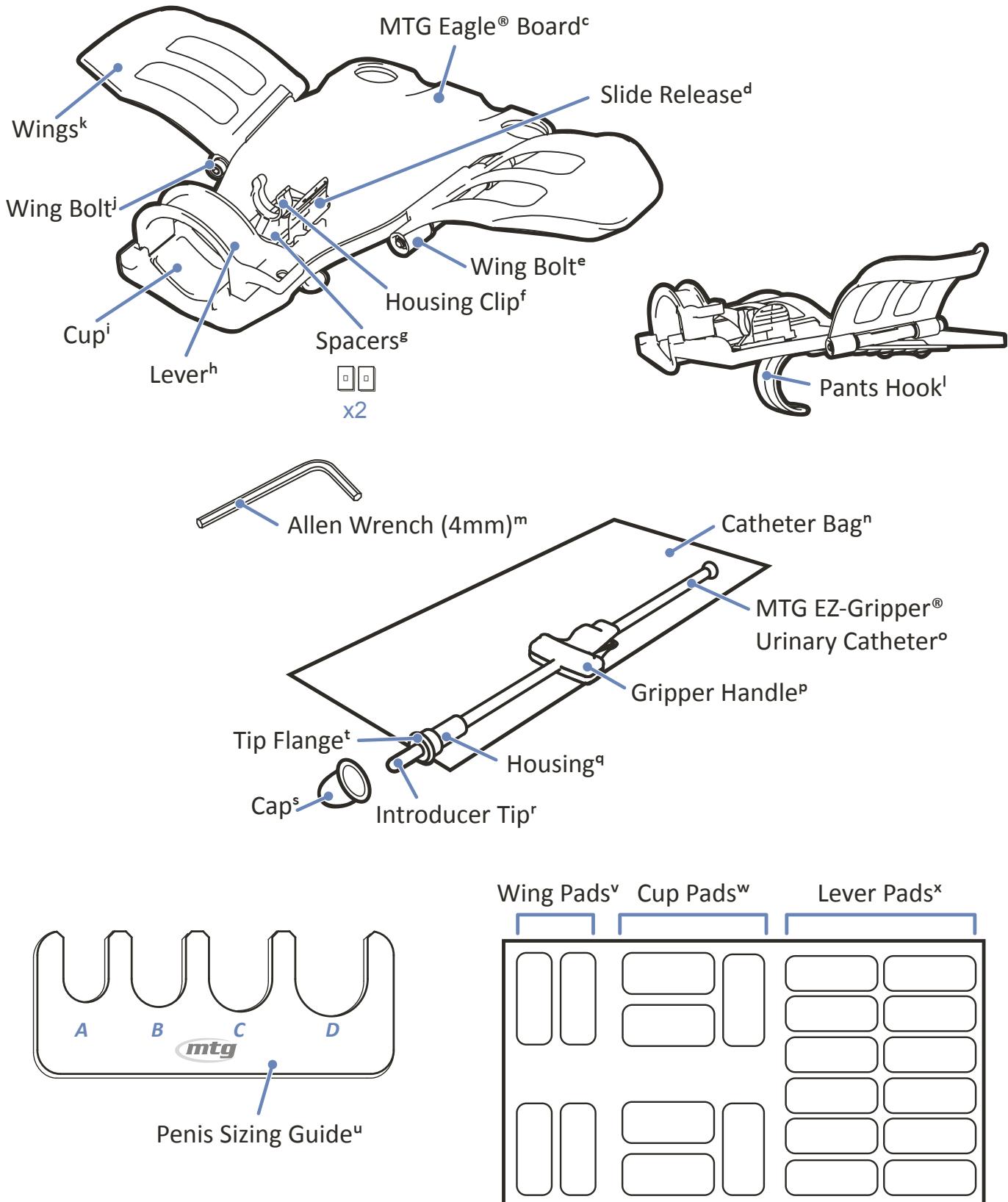

# Initial Set Up<sup>a</sup>


Before use, clean per instructions on page 9-10.<sup>b</sup>

It may be necessary that proper training be provided during the initial setup of the MTG Eagle® Board by a caregiver, healthcare provider, occupational therapist, or nurse assistant.<sup>c</sup>


Reference for padding.<sup>d</sup>


A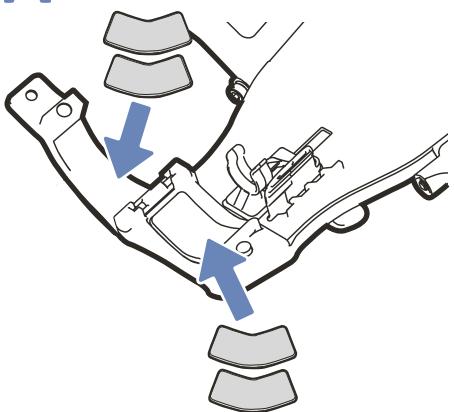



B

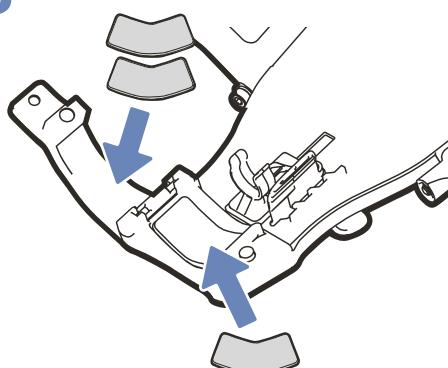C

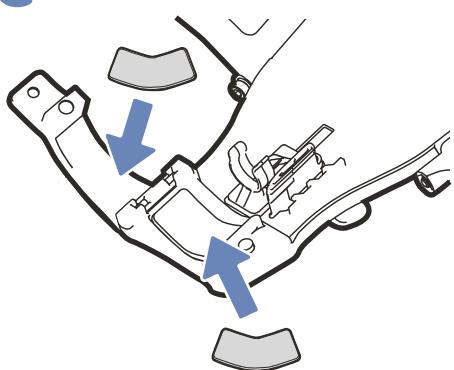
D

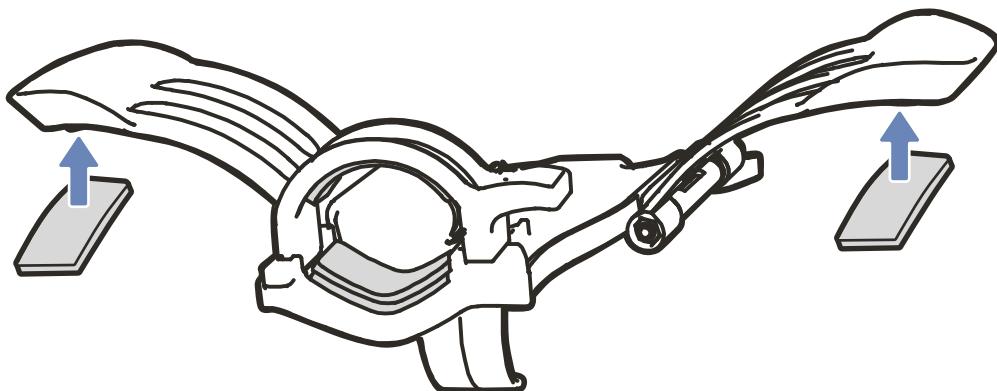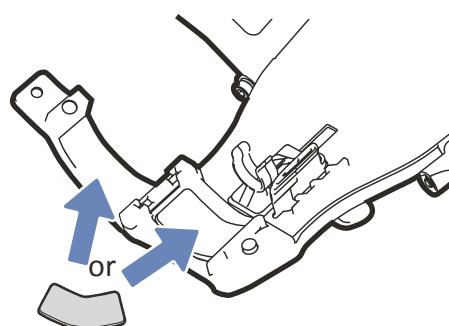
As needed, remove and replace the foam pads.<sup>e</sup>

Use spacers to adjust the height of the Housing Clip.<sup>a</sup>  
 The Introducer Tip of the MTG EZ-Gripper® should line up with the meatal opening of the penis.<sup>b</sup>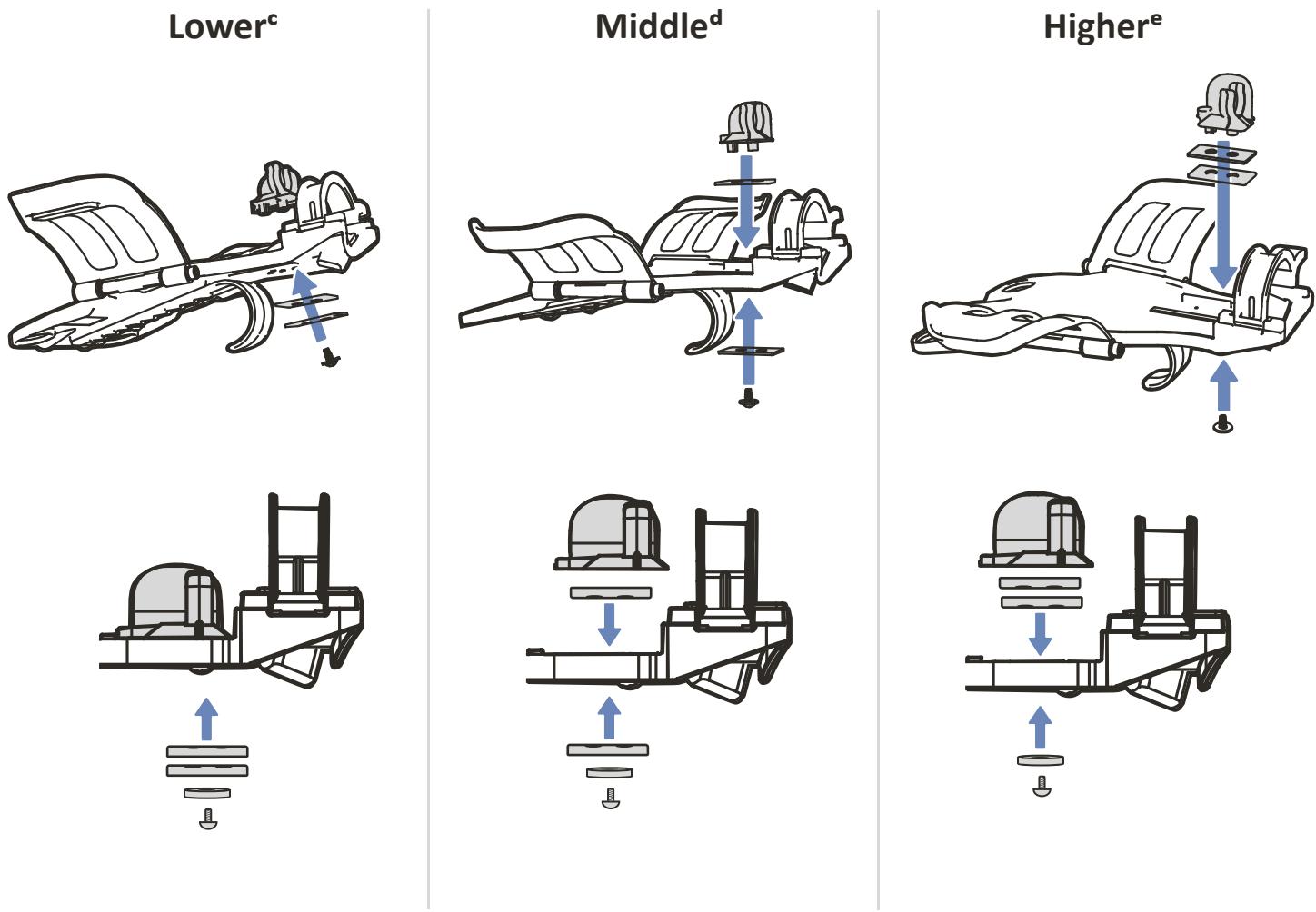


Use the Allen wrench to tighten or loosen the wing bolts, as needed.<sup>f</sup>  
 Note that tighter bolts will cause the wings to hold their position better if a specific angle is needed.<sup>g</sup>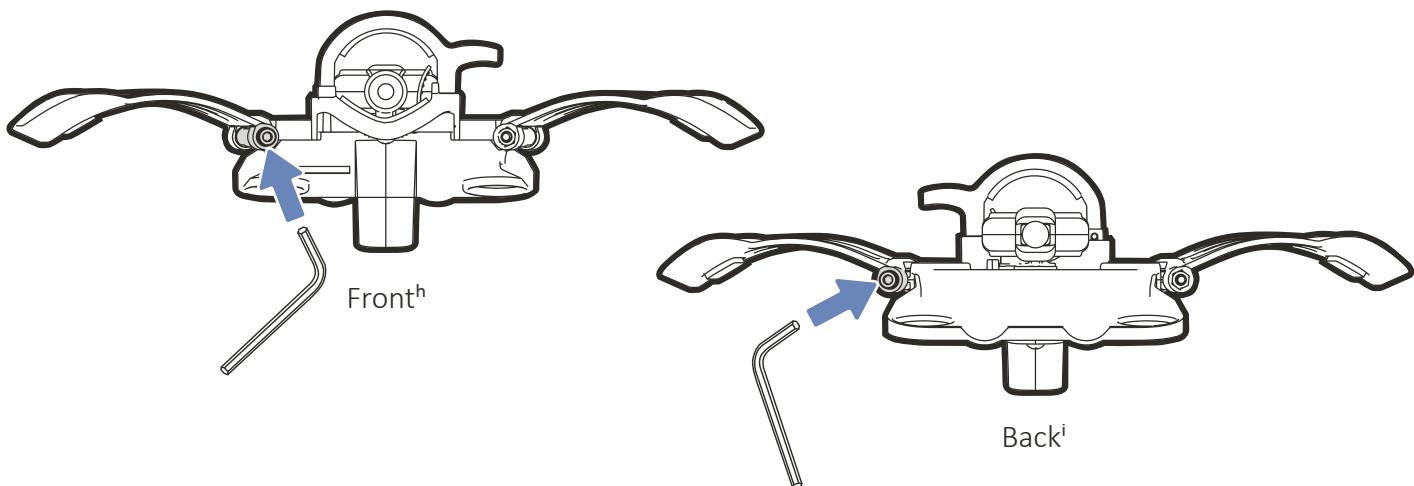



# Directions For Use<sup>a</sup>

⚠ Use on intact skin only<sup>b</sup>

1
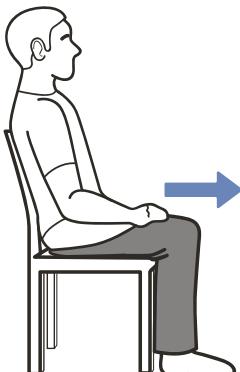


2

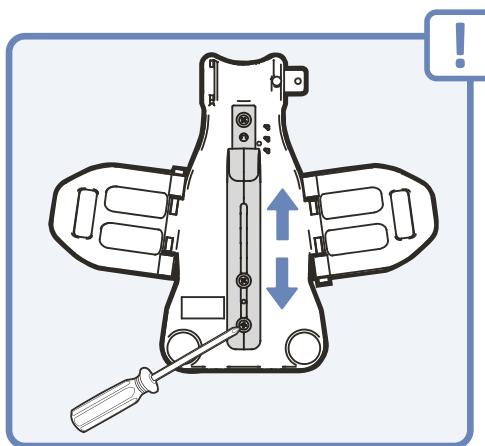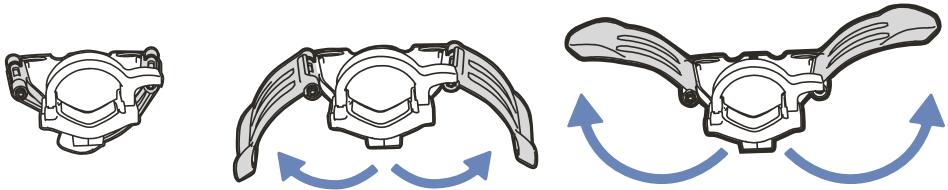3


3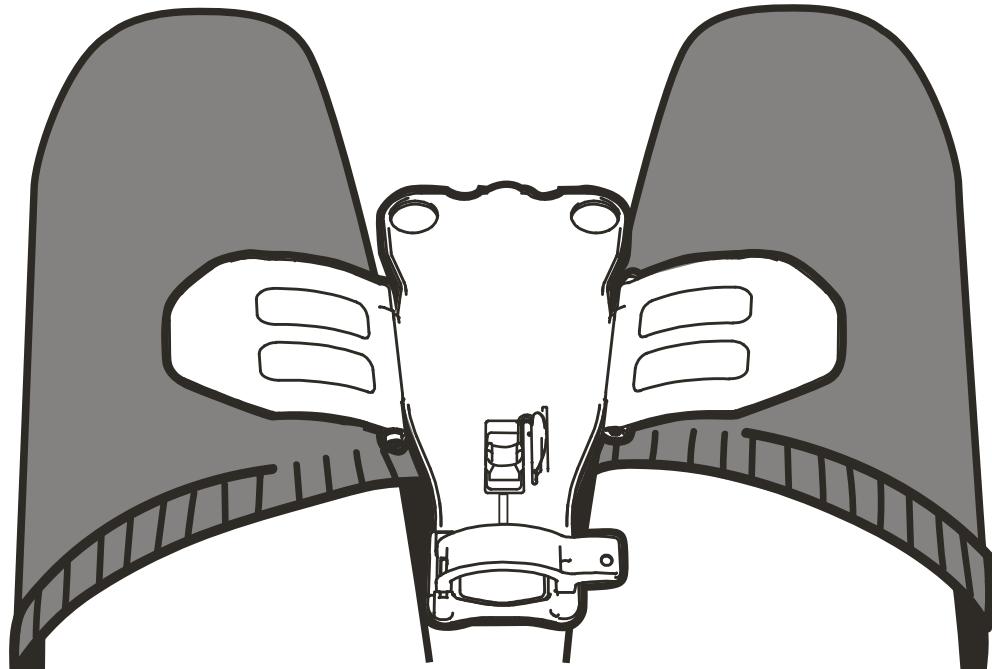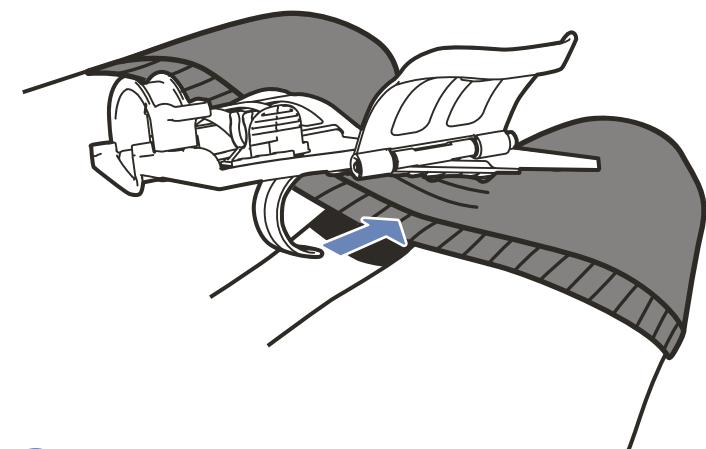

---

**4**





---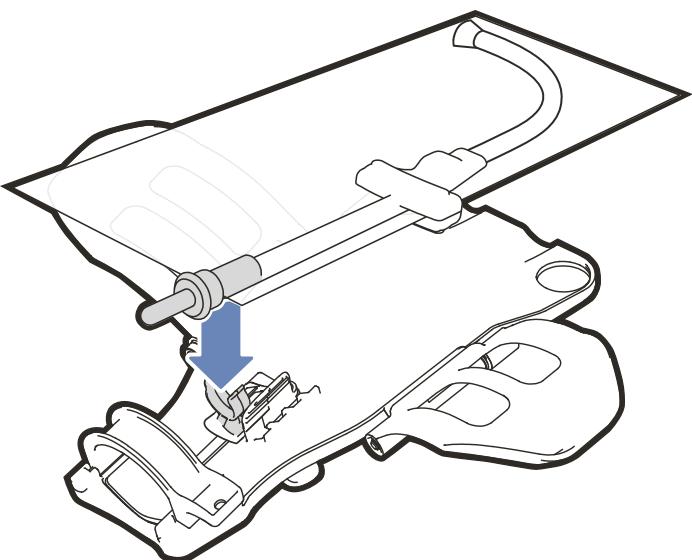

---

**5**

---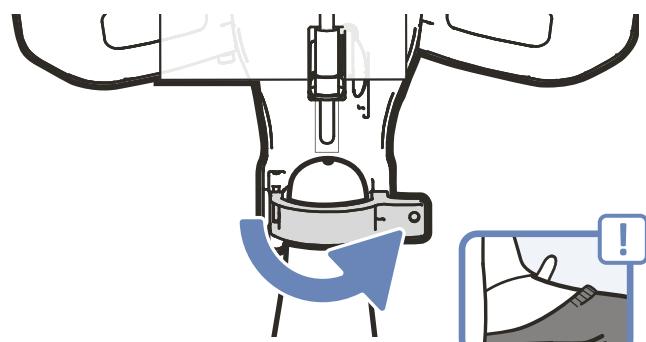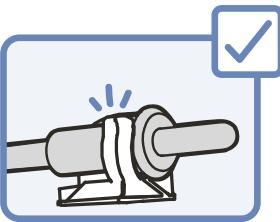
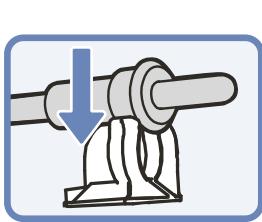
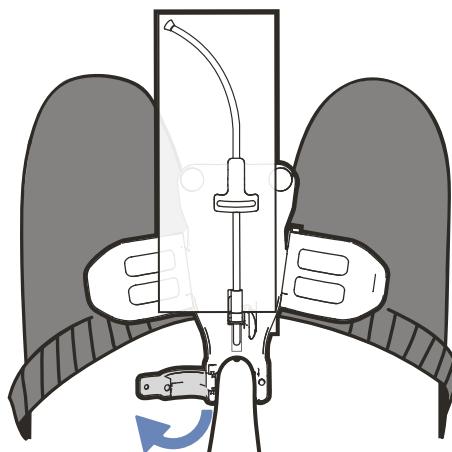


Refer to page 8.<sup>a</sup>

---

**6**

---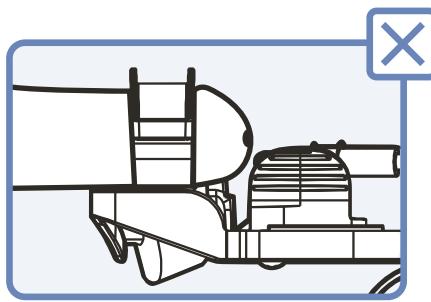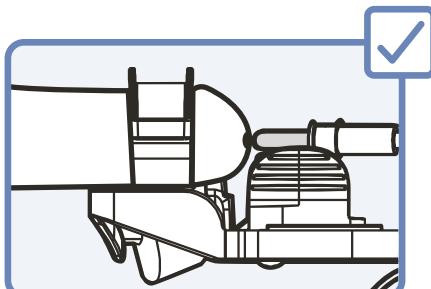
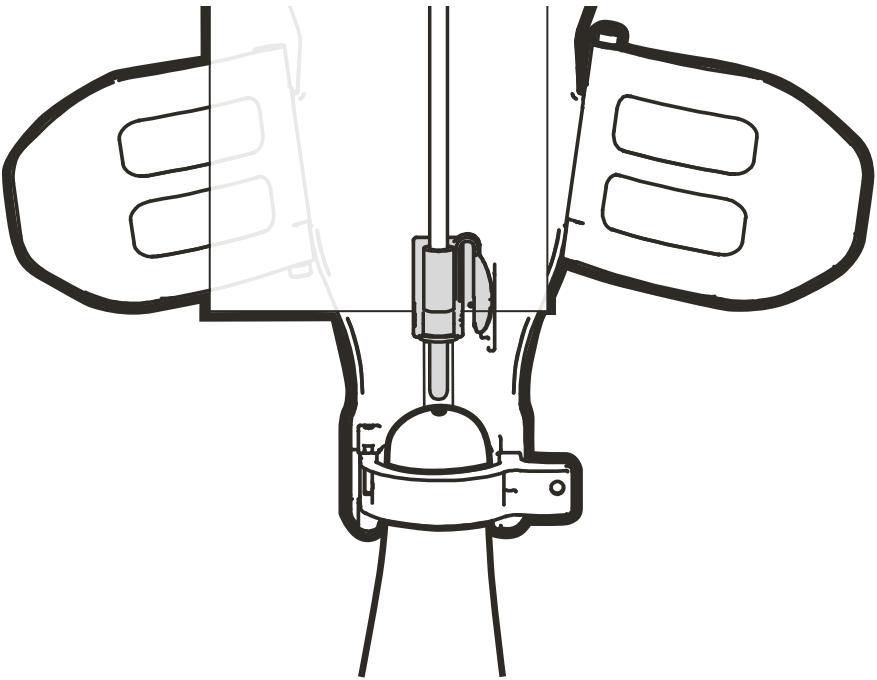


Refer to page 3.<sup>b</sup>

---

**7**



---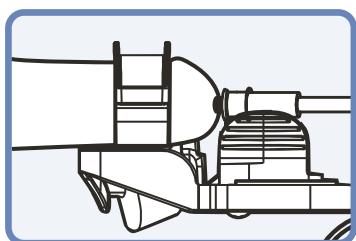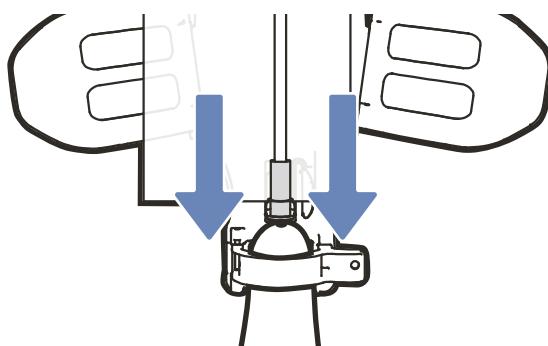

---

**8**





---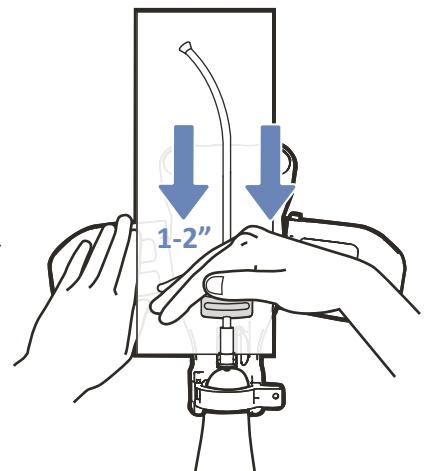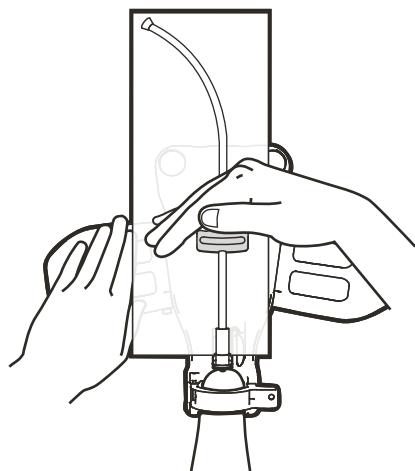

---

**9**


---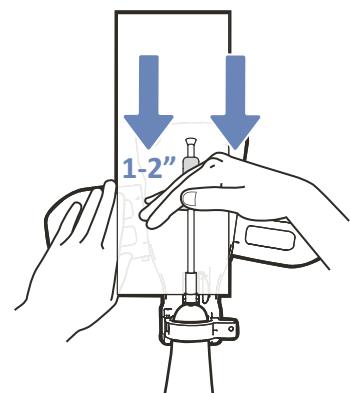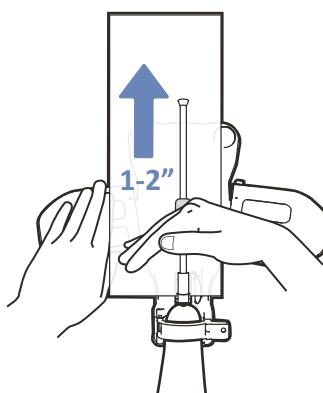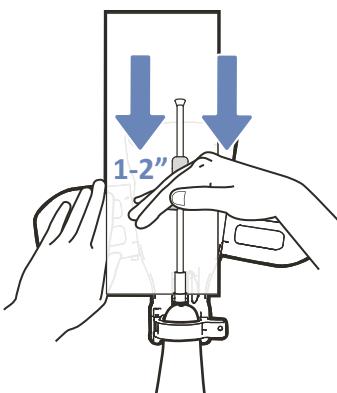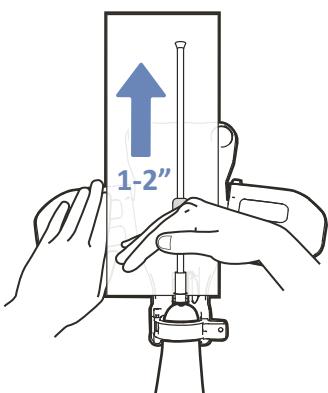

---

**10**

---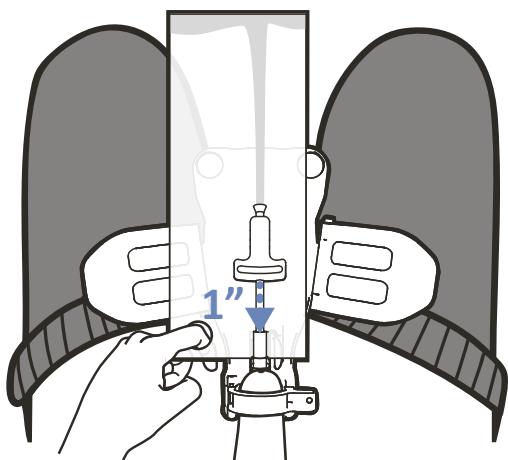


**NOTE:** When urine starts to flow, advance catheter up 1 more inch to ensure complete drainage.<sup>a</sup>

---

**11**


---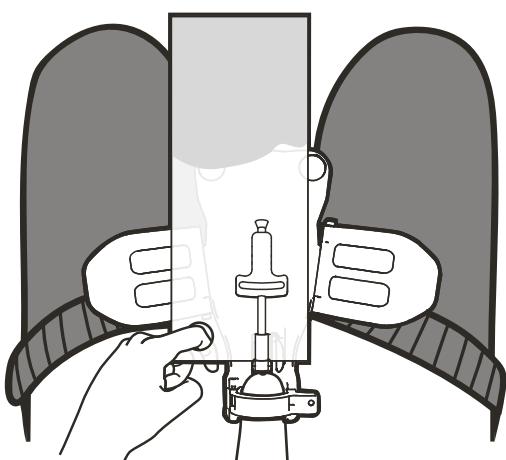

---

12


---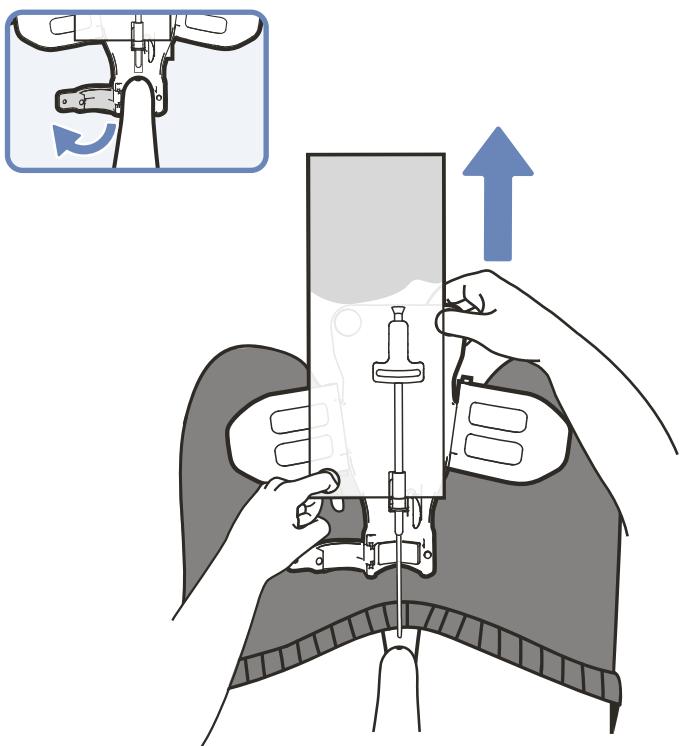

---

13



---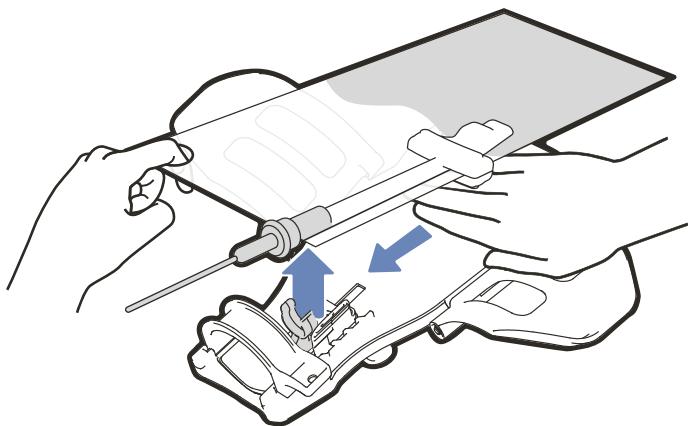


---

14

---

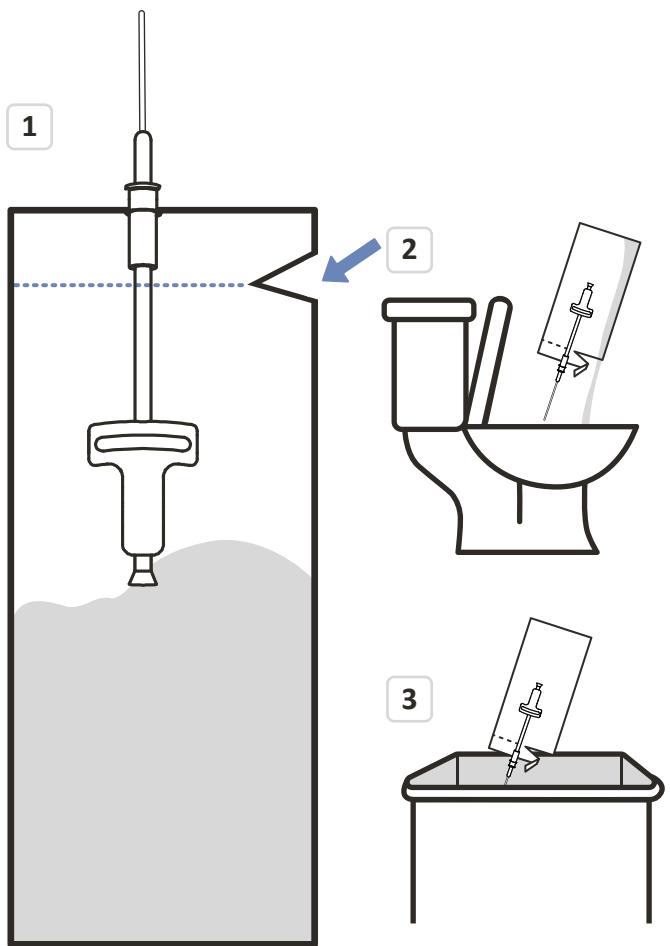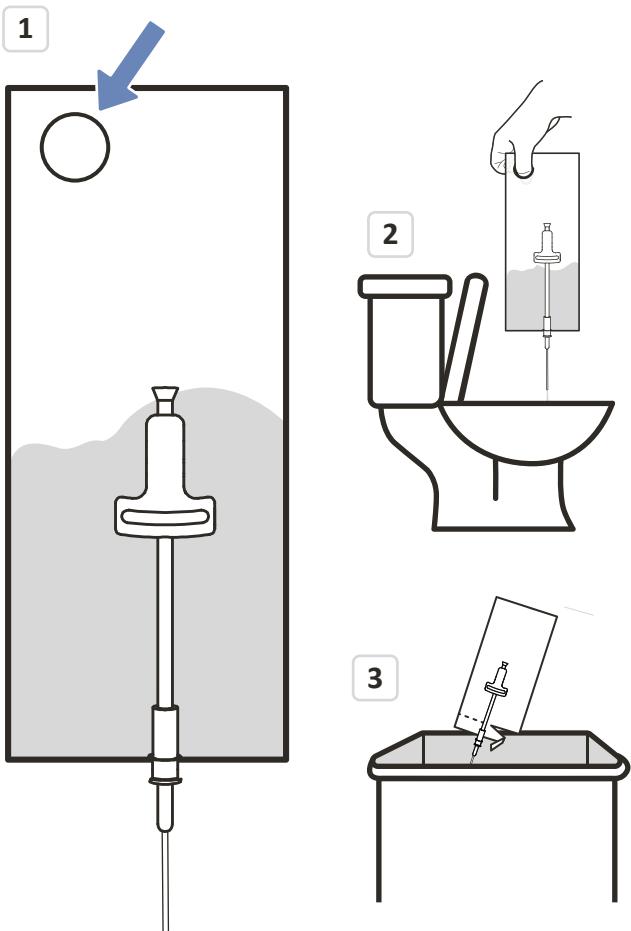OR<sup>a</sup>

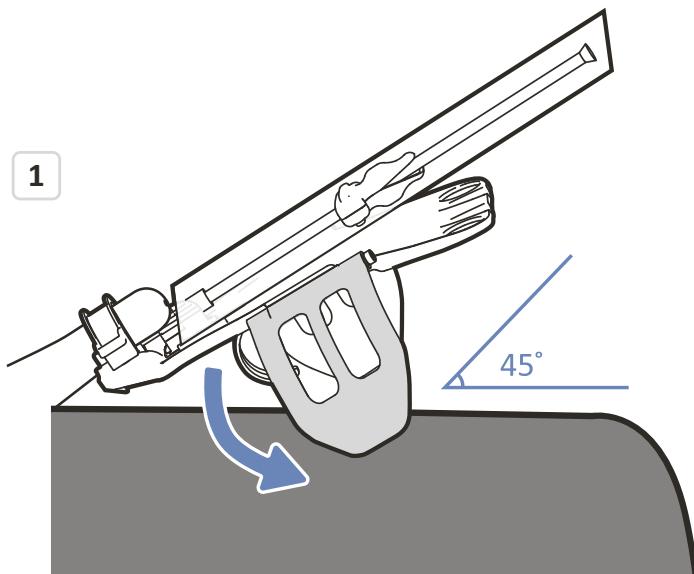
---


**Complete steps 6 - 10<sup>a</sup>**

---

When urine starts to flow, flatten the board.<sup>b</sup>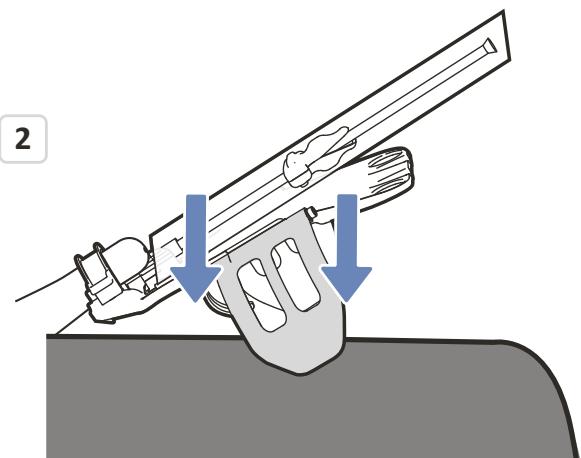

**3**

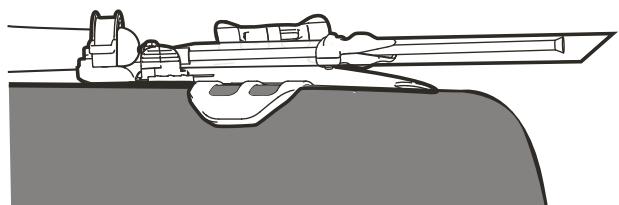---



**Complete steps 12 - 14<sup>c</sup>**

---

# Cleaning Instructions<sup>a</sup>


## Personal Use<sup>b</sup>

Before the first use of the MTG Eagle® Board and after each use, wash all surfaces of the board thoroughly with soap and water and towel dry.<sup>c</sup>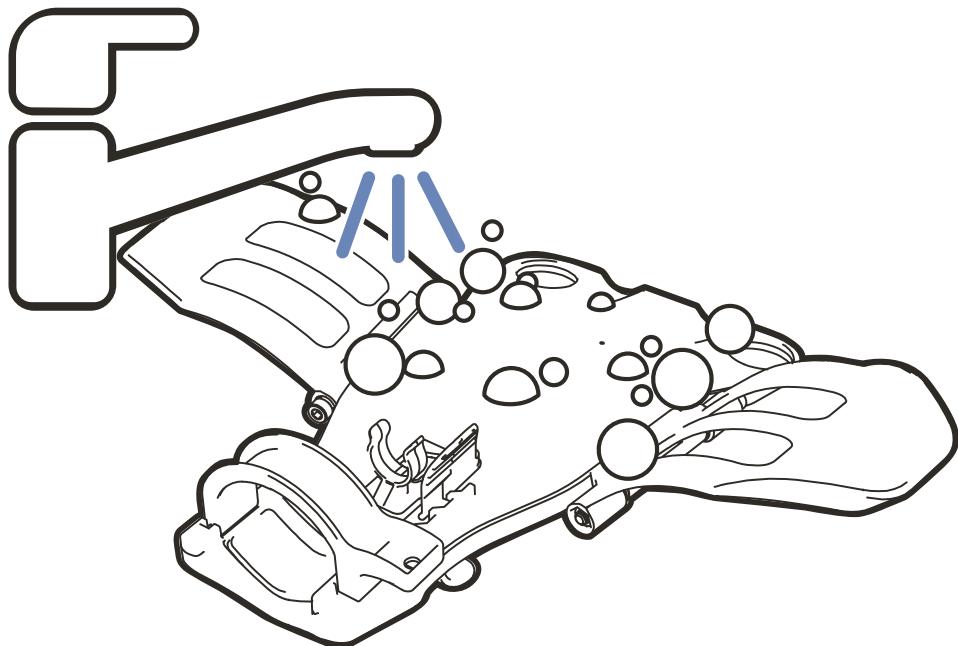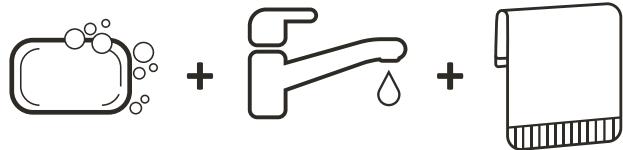


## Clinical Use<sup>a</sup>


Before the first use of the MTG Eagle® Board, wipe all surfaces of the board with an alcohol pad and let air dry.<sup>b</sup>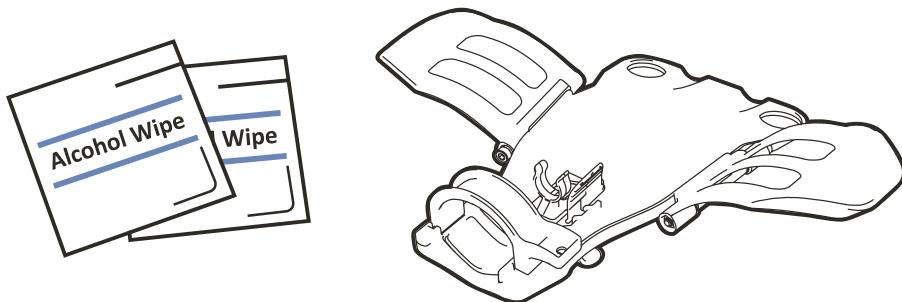

After each use by a different person, remove the foam pads and dispose of in appropriate waste container.<sup>c</sup>  
Wipe all surfaces of the board with an alcohol pad and let air dry.<sup>d</sup>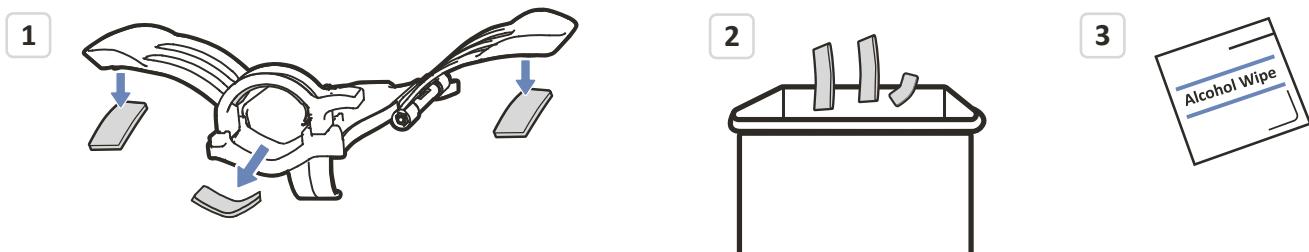

Replace the foam pads with new foam pads in preparation for the next person.<sup>e</sup>

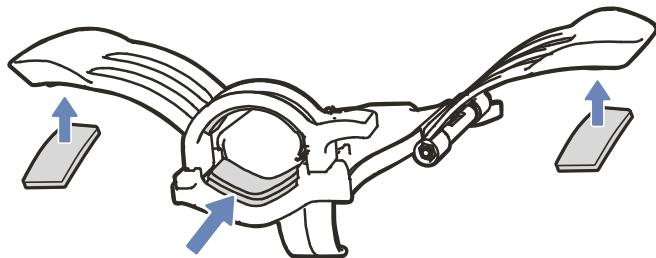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To watch user videos of our products, please visit our website at medtechga.com.

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## MTG Eagle® Board Instruction Manual

Medical Technologies of Georgia

15151 Prater Dr., Suite E,

Covington, GA, USA, 30014


01-(770)-788-0763

[www.medtechga.com](http://www.medtechga.com)

Nulla in commodo libero. Sed sed elit nulla, non enim. In hac habitasse nec purus velit. Phasellus fermentum enim et orci tempus laoreet. Ut erat, ultricies sem. Mauris commodo varius mattis. Mauris bibendum neque nec sem sagittis. Mauris tricies sem. Mauris auctor, ante euismod, ante consectetur eleifend imperdiet.

*In the event that any product sold by MTG is defective, unmerchantable or non-conforming, the total liability of MTG is expressly limited to the replacement of the defective product or a refund issued for the price paid by purchaser (at MTG's discretion) within 90 days from date of purchase. Replacement or refund for the allegedly defective product will be the sole and exclusive remedy of purchaser against MTG. For full MTG Eagle® Board return/exchange/warranty policy, please visit [www.medtechga.com](http://www.medtechga.com)*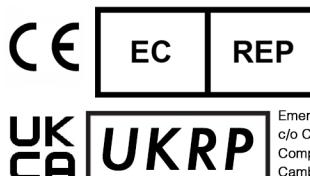

EMERGO EUROPE  
Prinsesegracht 20  
2514 AP The Hague  
The Netherlands

Emergo Consulting (UK) Limited  
c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ United Kingdom

PK-1298 REV03 08/21